ScanBox^{pro}

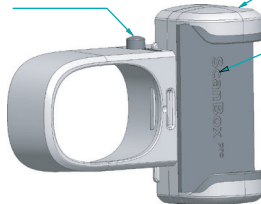
Phone support + Cheek retractor
Support pour téléphone + Tube rétracteur de joue
Telefonhalterung + Wangenhaltertubus
Soporte del teléfono + Tubo retractor de mejilla
Supporto per il telefono + Tubo divaricatore per guance
Suporte de telemóvel + Afastador labial em tubo
Telefoonsteun + Mondspreiderbuis
Uchwyt na telefon + Retraktor policzkowy
Puhelimen pidikkeen + Poskien levitinputken
Mobilhållare + Munvinkelrör
Držiak na telefón + Trubica na retrakciu líc
Držák telefonu + Trubkový retractor tváří

Instructions for use
Mode d'emploi
Gebrauchsanweisung
Instrucciones de uso
Istruzioni per l'uso
Instruções de utilização
Gebruiksaanwijzing
Instrukcja użytkowania
Käyttöohje
Bruksanvisning
Návod na použitie
Návod k použití

I.	Glossary / Glossaire / Glossar / Glosario / Glossario / Glossário Verklaring van de symbolen / Glosariusz / Sanasto Ordlista / Slovník / Význam symbolů	4	III.	Instructions for use Mode d'emploi Gebrauchsanweisung Instrucciones de uso Istruzioni per l'uso Instruções de utilização Gebruiksaanwijzin Instrukcja użytkowania Käyttöohje Bruksanvisning Návod na použitie Návod k použití	14 22 30 38 46 54 62 70 78 86 94 102
II.	Description / Description / Beschreibung Descripción / Descrizione / Descrição Beschrijving / Opis / Kuvaus Beskrivning / Opis / Popis	6			
		2	3		

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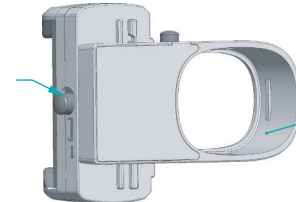
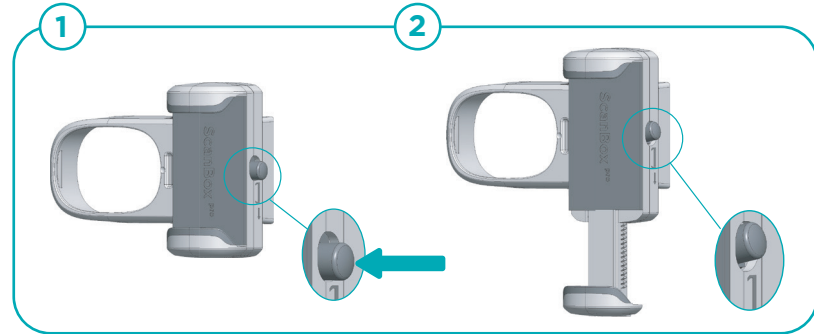
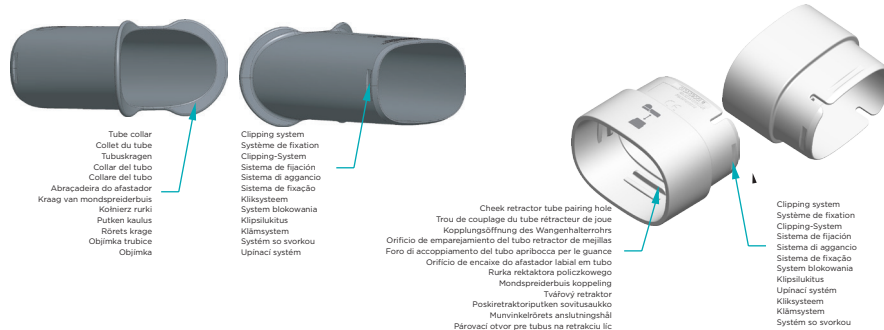
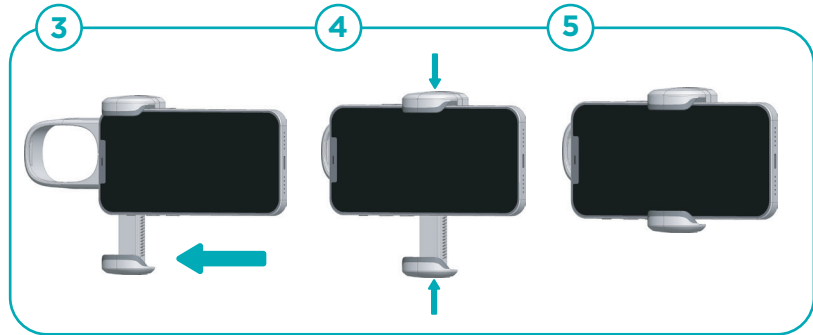
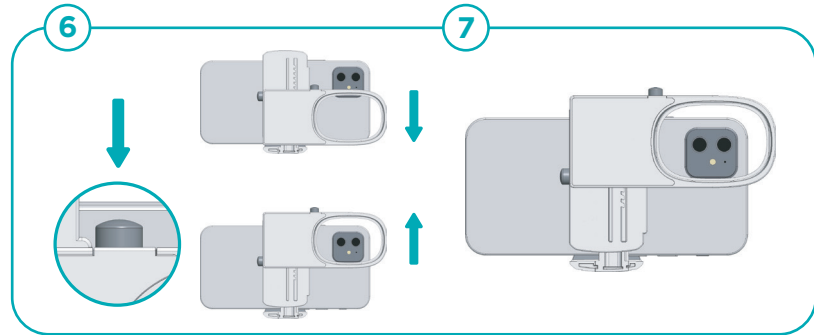
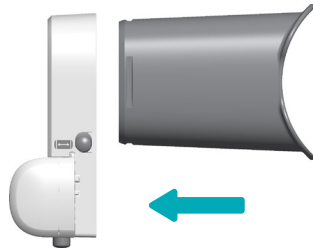
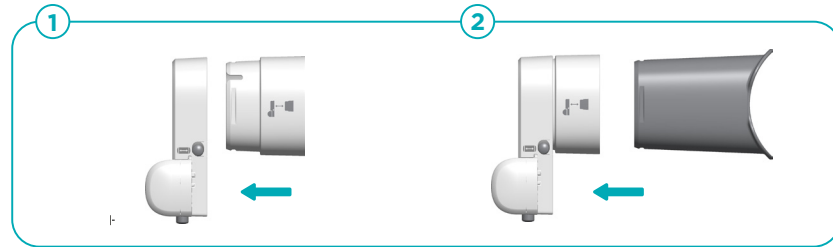
PRESENTATION

2 packs of ScanBox^{pro} are available:

- **ScanBox^{pro} S-M pack** (REF 1009-001) containing one **Phone support** (REF 1005-001), one **Cheek Retractor Tube Size S** (REF 1006-001), one **Cheek Retractor Tube Size M** (REF 1007-001), one Adapter (REF 1013-001) and one mesh pouch.
- **ScanBox^{pro} M-L pack** (REF 1010-001, REF 1012-001) containing one **Phone support** (REF 1005-001), one **Cheek Retractor Tube Size M** (REF 1007-001), one **Cheek Retractor Tube Size L** (REF 1008-001), one Adapter (REF 1013-001) and one mesh pouch.

INTENDED PURPOSE

ScanBox^{pro} retracts the cheeks to allow a full view of a patient's mouth to take pictures of the intraoral cavity.

INDICATIONS

ScanBox^{pro} is intended to be used as part of a scanning procedure guided by softwares, developed, and marketed by DentalMonitoring, to monitor intraoral health and orthodontic treatments.

TARGET GROUPS

- **Target population: ScanBox^{pro} is indicated for patients over the age of 6.**
- **Intended users: ScanBox^{pro} is intended to be used by:**
 - Healthcare professional in professional healthcare facilities, such as a dental practice.
 - Lay persons over the age of 6 in a non-healthcare environment, such as the patient's own home.

- **Type of use/reuse: ScanBox^{pro}** is reusable and can be used:
 - Multiple times by a single patient.

CLINICAL BENEFITS

There is no direct clinical benefit for the use of **ScanBox^{pro}**. **ScanBox^{pro}** is intended to be used to ease the taking of scans that will be processed by DentalMonitoring softwares.

CLINICAL PERFORMANCES

ScanBox^{pro} has no clinical performances as it has no direct action and effect on the patient.

TECHNICAL SPECIFICATIONS

ScanBox^{pro} biocompatibility was assessed, and the product was demonstrated to be non-cytotoxic, non-sensitizing and non-irritating.

SIDE EFFECTS

No known side effects.

CONTRAINDICATIONS

ScanBox^{pro} should not be used in the event of injured skin or injured mucous membrane.

ScanBox^{pro} is not intended for surgical or post-surgery use, nor to take intra-oral pictures with products other than those marketed by DentalMonitoring.

WARNINGS, PRECAUTIONS, NOTICES

• Warnings

No known warnings.

• Precautions

ScanBox^{pro} is provided in a non-sterile form. Clean the **ScanBox^{pro}** before and after each use. Please refer to the appropriate cleaning procedure to avoid any health issue requiring medical attention.

Do not clean with household cleaning products.

The **Phone support** and the Adapter are not sterilizable and do not submerge it in any liquid. Refer to the cleaning procedure to avoid any contamination.

The following population should be assisted by a third party in device use:

- children up to 12;
- visually impaired adults or children;
- hearing impaired adults or children;
- adults or children with any other condition that might prevent the patient from adequately operating the device.

For patients with deciduous teeth, the guardian's vigilance is required to ensure that deciduous teeth do not fall out when placing the **Cheek Retractor Tube**.

If your smartphone is not correctly held in position when the jaw of the **Phone support** is tightened, do not force, this could damage your smartphone or the device. If you haven't already done it, remove the protective cover from your phone and try again.

Do not use the **ScanBox^{pro}** if it is broken.

Do not introduce the Adapter into the mouth.

• Notices

This is not a toy. Children should be supervised to ensure that they do not play with the device.

INSTRUCTIONS BEFORE USE

Before use, you should wash your hands with soap and water or hydroalcoholic gel. You must also clean the **ScanBox^{pro}** and your smartphone (following its manufacturer's recommendations) before each use.

Refer to the cleaning procedure for the appropriate steps to follow.

PAIRING UP THE PHONE SUPPORT WITH YOUR PHONE

For the numbering below, please refer to the **Phone support** operating procedure schematics.

Launch the application you want to use on your phone before using the **ScanBox^{pro}**.

Hold the **Phone support** with the jaw's engraved pad facing you (1). Open the jaw by pressing the jaw's opening button (2). Place your phone against the fixed jaw and slide it horizontally to align the flash and lens with the tube pairing hole (3).

Clamp the jaw onto your phone by pressing firmly on the mobile jaw (4). Remove the protective cover of your phone if necessary.

Make sure that your phone is held firmly (5).

Move your phone vertically so that the flash and lens are in the tube pairing hole.

To do this, hold down the tuning button and move the jaw vertically (6).

Release the tuning button when you are finished (7).

PAIRING UP THE PHONE SUPPORT WITH THE CHEEK RETRACTOR TUBE

When your phone is placed correctly, connect the **Cheek Retractor Tube** to the **Phone support** by inserting the end of the tube with the clipping system into the tube pairing hole on the device.

You will hear a "click" when the **Cheek Retractor Tube** is properly attached.

PAIRING UP THE PHONE SUPPORT WITH THE ADAPTER AND THE CHEEK RETRACTOR TUBE - ONLY IF THE APPLICATION REQUEST YOU TO USE THE ADAPTER

The adapter may be required depending on your phone model; the application automatically detects whether the adapter is required and informs you accordingly.

When your phone is placed correctly, connect the **Adapter** to the **Phone support** by inserting the end of the **Adapter** with the clipping system into the tube pairing hole on the phone support (1). You will hear a “click” when the **Adapter** is properly attached.

Then, connect the **Cheek Retractor Tube** to the **Adapter** by inserting the end of the tube with the clipping system into the **Adapter** pairing hole (2). You will hear a “click” when the **Cheek Retractor**

Tube is properly attached.

INSERTING THE CHEEK RETRACTOR TUBE IN YOUR MOUTH

Appropriate sizing should be determined within a healthcare facility.

If not possible, first try to use the biggest size of **Cheek Retractor Tube** available. If you feel discomfort or pain, change for the smaller size.

Once the **ScanBox^{pro}** is assembled, hold the device without touching the collar that goes in the mouth with your hands. The screen of your phone needs to be facing out.

First, insert one side of the tube collar into your mouth and then the other. Make sure that the collar is inserted correctly in your mouth: the tube must not escape from your mouth when moving it laterally. Check if the tube is correctly applied: teeth and

gums should be visible without experiencing pain or discomfort.

In the event of significant pain or discomfort, refer to your healthcare facility to check you have the appropriate tube size.

UNPAIRING THE PHONE SUPPORT AND THE CHEEK RETRACTOR TUBE

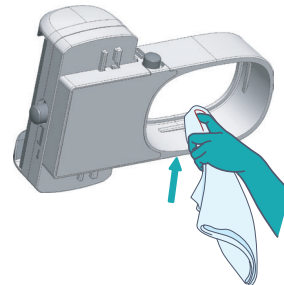
Remove the **Phone support**, by holding the **Cheek Retractor Tube** in place with your hand and pulling the device forward.

INSTRUCTIONS FOR REUSABLE DEVICES

The following cleaning procedure must be followed:

- **Clean the smartphone camera lens**, if needed, following its manufacturer’s recommendations.

CLEANING

- **Clean the Phone support:**

Before and after each use, clean the whole Phone support (see the picture below) by using a biocide wipe or a biocide spray paired with single-use paper towels.

Let the **Phone support** air dry.
Then store in the provided mesh pouch.

The **Adapter** shall follow the same cleaning procedure as the phone support.

- **Clean the Cheek Retractor Tube**

Before and after each use, clean with soap, such as Sanex 0%, and cold water.

If necessary, gently brush the surface with a soft bristle brush and make sure it is completely clean.
Rinse with cold water.

Let the **Cheek Retractor Tube** air dry.

Then store it in the provided mesh pouch, in a dry and clean place, avoiding compression of the **Cheek Retractor Tube** with other objects.

INSTRUCTIONS FOR DAMAGED PACKAGING

If the packaging is damaged or opened before use, return the pack to your healthcare facility.

INSTRUCTIONS FOR VERIFICATION

The device must be properly assembled before use.

STORAGE

The **ScanBox Pro** should be stored in the provided mesh pouch, away from solar radiation, compression and moisture. Keep away from any heat source.

DISPOSAL

Dispose of it in accordance with your local recycling regulations.

LIST OF REQUIRED NON-PROVIDED PRODUCTS

DentalMonitoring (REF 2001-001), SmileMate (REF 2002-001)

WARRANTY

One year from the date of original purchase.

TRAVEL

In the event of travel, the devices should be kept in the provided mesh pouch.

Please make sure the devices are not too tightly packed to prevent any deformation or breakage during transport.

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